

**F&P InfoSmart Web™** is a simple, cost-effective solution that allows your CPAP progress to be monitored by your healthcare provider.



INFOUSB IS AVAILABLE FOR THE FOLLOWING PRODUCTS:



F&P ICON™/ICON™+ CPAP Family

InfoUSB is used as an accessory to the following: F&P ICON/ICON+, Sleepstyle™ 200 Auto and Sleepstyle 200 (HC242/HC244) series of CPAP devices.

The InfoUSB allows CPAP compliance and efficacy data to be transmitted over the Internet via a USB port on a computer. This information is sent to F&P InfoSmart Web, and allows for review by the clinician and remote adjustment of the CPAP device settings.



User Guide

Upload data on:

..... /..... /.....

CE 0123

**Manufacturer** Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348, Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.com  
**Australia** Fisher & Paykel Healthcare Pty Limited, 36-40 New Street, PO Box 167, Ringwood, Melbourne, Victoria 3134, Australia Tel: +61 3 9879 5022 Fax: +61 3 9879 5232 **Austria** Tel: 0800 29 31 23 Fax: 0800 29 31 22 **Benelux** Tel: +31 40 216 3555 Fax: +31 40 216 3554 **China** Tel: +86 20 3205 3486 Fax: +86 20 3205 2132 **France** Tel: +33 1 64446 5201 Fax: +33 1 64446 5221 **Germany** Tel: +49 7181 98599 0 Fax: +49 7181 98599 66 **India** Tel: +91 80 4284 4000 Fax: +91 80 4123 6044 **Irish Republic** Tel: 1800 409 011 **Italy** Tel: +39 06 7839 2939 Fax: +39 06 7814 7709 **Japan** Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea** Tel: +82 2 6205 6900 Fax: +82 2 6309 6901 **Northern Ireland** Tel: 0800 132 189 **Spain** Tel: +34 902 013 346 Fax: +34 902 013 379 **Sweden** Tel: +46 8 564 76 680 Fax: +46 8 36 63 10 **Switzerland** Tel: 0800 83 47 63 Fax: 0800 83 47 54 **Taiwan** Tel: +886 2 8751 1739 Fax: +886 2 8751 5625 **Turkey** Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, Alinteri Bulvarı 1161/1 Sokak No. 12-14, P.O. Box 06371 Ostim, Ankara, Turkey Tel: +90 312 354 34 12 Fax: +90 312 354 31 01 **UK** Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: +44 1628 626 136 Fax: +44 1628 626 146 **USA/Canada** Tel: 1800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001

REF 185046915 REV G 2014-11 © 2014 Fisher & Paykel Healthcare Limited



www.fphcare.com

**Fisher & Paykel**  
HEALTHCARE



### Step 1

#### INSERT INTO COMPUTER

When requested by your healthcare provider, remove the InfoUSB from your CPAP device and insert it into the USB port of a computer.

A small light illuminates when connected to your computer. If the light does not illuminate, please turn the stick around.

### Step 2 (Mac)

#### INSTALL INFOUSB APPLICATION

Launch the Mac App Store and search for **InfoUSB**. Install this free application. Upon successful installation, open the Launchpad and run InfoUSB.

*A Mac running OSX 10.8 or later and an Internet connection are required.*

### Step 2 (Windows)

#### INSTALL INFOUSB APPLICATION

##### Windows 8:


Launch the Windows App Store and search for **InfoUSB**. Install this free application. Upon successful installation, open the app and run InfoUSB.

*A PC or tablet running Windows 8 and an Internet connection are required.*

##### Windows PC:

Click on the Start button and open My Computer. Navigate to the drive called FPHCARE. Open this folder and double-click on the **Setup.exe** file. Follow the on-screen instructions. Upon successful installation of the InfoUSB application, the message in Step 3 will appear.


*A PC running Windows XP or later and an Internet connection are required.*

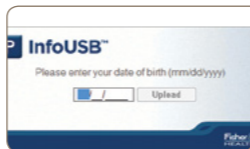

### Step 3

#### DATA TRANSFER

Upon detection of an InfoUSB in your computer, the above message will appear. Enter your Date of Birth and select the Upload button. Ensure that your computer is connected to the Internet for successful data transfer to your healthcare provider.

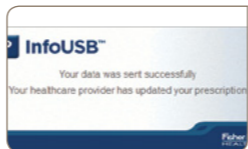

### Step 4

#### CONFIRMATION

After the data has been sent successfully, the confirmation message above will appear. If your prescription is updated, you will also see the message "Your healthcare provider has updated your prescription".



### Step 5

#### FUTURE LOGGING

Remove the InfoUSB from your computer and place it back into your CPAP device. The CPAP device is now ready to be used for continued therapy.

The next time you need to transfer your CPAP data to your healthcare provider, simply insert the InfoUSB into your computer and the message in Step 3 will automatically appear.

## FAQs

***I tried to upload my sleep data but it was unsuccessful. Why did this happen?***

There are a number of reasons why your sleep data may not have uploaded successfully.

1. **Your computer was not connected to the Internet.**

Ensure you are connected to the Internet before attempting to upload your sleep data.

2. **Your InfoUSB does not contain any data.**

Data will only transfer to the InfoUSB if it was correctly inserted into the CPAP device. Try re-inserting the InfoUSB into your CPAP device to transfer the data.

3. **Your CPAP details were incorrectly entered or you are not correctly set up in F&P InfoSmart Web.**

Contact your healthcare provider to check that you have been set up correctly in F&P InfoSmart Web.

***How often do I need to send my sleep data?***

Your healthcare provider will let you know when to upload your data through F&P InfoSmart Web.